

Maintenance Log

Maintenance Log for AED

Check AED is in a visible and unobstructed location

Inspect AED condition for damage and wear

Check the batteries:

Correct installation

Valid expiration date

Replace expired batteries

Status/service indication light working

Check the electrodes/AED pads

Valid expiration date

Replace AED pads if expired

Ensure cables from electrodes are plugged in

Check for adequate supplies (mask, gloves, razor, moist towlette, towel, extra batteries)

Refer to the manufacturer for more information and proper maintenance procedures for your particular AED Model

Date/Time Notes

Date/Time Notes

Date/Time Notes

Date/Time Notes

Date/Time Notes